# TEL CARE®

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Blood Glucose Monitoring System

# Quick Reference Guide


Before first use, charge the glucose meter for at least 3 hours with the AC charger included in your kit. Follow control solution instructions on Page 8 of the BGM User Manual.

Clean and disinfect your lancing device after every use. Clean and disinfect your meter at least once a week. PLEASE REFER TO DETAILS IN THE MANUAL.

1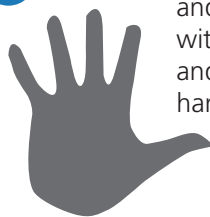

Wash your hands and testing site with warm water and soap. Dry your hands well

2


Turn on the glucose meter using the power button. Slight delays may occur when meter powers on.

3


When the **Insert Test Strip** screen appears, insert the Telcare test strip. Continue to push the strip in until **Apply Blood** appears.

5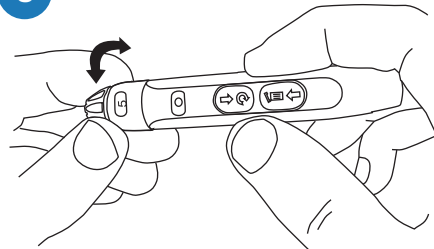

Adjust the cap to your desired puncture depth level.

4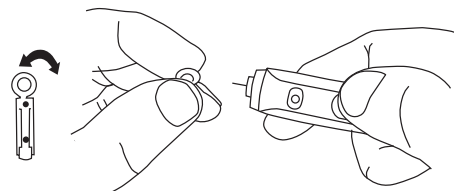


Insert a new lancet into the lancing device and twist off its plastic cap.

**IMPORTANT:**


The lancing device is intended for single patient use.

6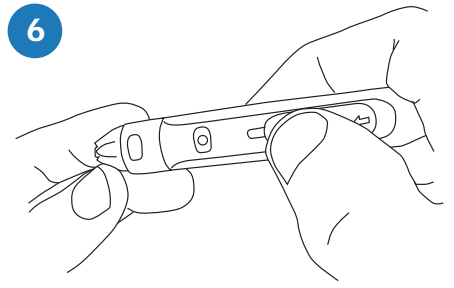

Using your thumb, push the sliding button back. You will hear a click, indicating that the lancet carrier is locked into position.

7

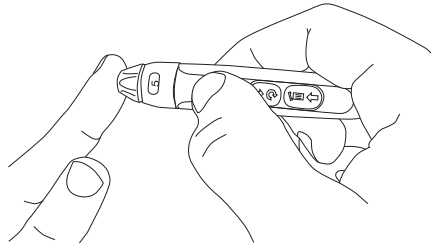Position the end of the device against the side of the fingertip. Press the trigger button to actuate the lancet.

9

Your test result will display.

8

Make sure the **Apply Blood** screen is displayed. Touch the tip of the test strip to the blood drop until the end is filled.

10

Middle  
Down

(Please refer to User Manual page 9 for button location.)

Highlight a reading tag using the **down** button. Enter the reading tag by pressing the **middle** button.

11


Remove test strip to send data to MyTelcare.com.

IF NO DATA CONNECTION:

1. Move to different location to get a cell signal,
2. Go to **History**,
3. Highlight unsent reading,
4. Press middle side button to send.

(Images are for reference only.)  
ART – 048 Rev. 3

12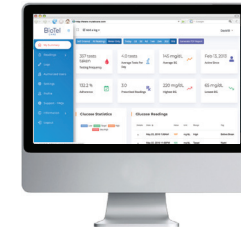

Log in to MyTelcare.com to:

- View blood glucose readings
- Invite caregivers
- Set up messaging options
- Create target zones
- Print data

See your Telcare BGM User Manual and Telsolve User Manual located at MyTelcare.com for full instructions.

Call 888-666-4530 (US) or  
(1) (978) 610-4688 (International)  
(9AM - 9PM EST, M-F)

with any questions or comments.  
For In Vitro Use Only

TEL CARE®

Telcare, LLC  
150 Baker Avenue Extension  
Suite 300  
Concord, MA 01742, USA