**Instructions inside. Open to learn more.**

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# Take your first test today!

If this is your first time using the device and it has not already been registered, register your device at <https://registration.mytelcare.com/>

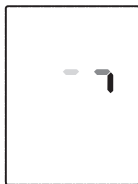
## 1. Installing the Batteries


**If this is your first time using the device:**

1. Open the battery cover on the bottom of your scale.
2. Install the batteries provided with the device.  
Follow the diagram inside the battery compartment for correct polarity - the springs should align with the negative side on the batteries.
3. Replace the battery cover until closed and secure.

## 2. Taking a Measurement

Transmission  
in progress

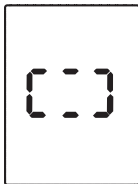Transmission  
successful

1. Place the scale on a flat and hard surface.
2. Step on the scale and remain still until the measurement is complete.
3. The device automatically transmits the measurement results. During this process, an indicator is shown in the middle of the display.
4. When the measurement is successfully transmitted, the device shows the following indication before turning off.
5. Log into your portal to see your results today!

**This page intentionally left blank.**

**This page intentionally left blank.**

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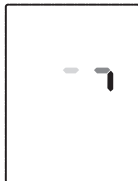
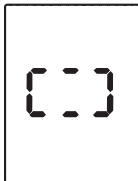
The BioTel Care® Terms of Service (“Terms”) outline the terms regarding your use of our products and can be found at <http://Biotelcare.com/terms>. If you do not agree with these Terms, do not register or use any of the Services.

Telcare, LLC  
600 West Ridge Road, Suite 200  
Linwood, PA 19061, USA  
Teléfono: (888) 666-4530 (EE.UU.)  
o (1) (978) 610-4688 (internacional)

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