

The Philips logo is displayed in the top left corner. It consists of the word "PHILIPS" in a bold, blue, sans-serif font. The text is contained within a white rectangular area that has a rounded bottom-right corner. A thick blue horizontal bar is positioned below the white area, extending from the left edge and curving slightly upwards at the right end.

**PHILIPS**

**Instructions included**

# Take your first test today

## Installing the batteries

If this is your first time using the device:

- 1 To remove the back cover compartment, push the white button and follow the direction of the printed arrows.
- 2 Install two AAA batteries into the battery compartment. Match the plus (+) and minus (-) signs in the compartment. If the polarities are not matched, damage to the oximeter may occur.
- 3 Slide the battery door cover horizontally in the direction of the arrow.



## Taking a measurement

- 1 Press the Power/Function button.
- 2 Place one finger into the rubber hole of the oximeter (it is best to plug the finger thoroughly) before releasing the clamp, with the nail upward.
- 3 The data (SpO<sub>2</sub> and Pulse Rate) will appear on the device. Don't move the finger; be sure to remain motionless during the reading.

Once the measurement is finished, it will display "Thank you". Remove your finger. The device will turn off automatically.

- 5 Log into your portal to see your results today.

## **Distributed by:**

Telcare, LLC

600 West Ridge Road Suite 200

Linwood, PA 19061

## **Phone:**

(888) 666-4530 (US /EE.UU.)

(1) (978) 610-4688 (International)

Monday - Friday, 9AM - 9PM EST

## **Email:**

VCM-customer@philips.com

[www.philips.com/virtualcaremanagement](http://www.philips.com/virtualcaremanagement)

The Terms of Service (“Terms”) outline the terms regarding your use of our products and can be found at [www.philips.com/virtualcaremanagement/terms](http://www.philips.com/virtualcaremanagement/terms). If you do not agree with these Terms, do not register or use any of the Services.

© 2023 Koninklijke Philips N.V. All rights reserved.