The Philips logo is displayed in a white square with rounded corners and a blue shadow. The word "PHILIPS" is written in a bold, blue, sans-serif font. A thick blue horizontal bar is positioned below the square, with a curved end on the right side.

PHILIPS

Instructions included

Instructions


Take your first test today

Installing the batteries


If this is your first time using the device:

- 1 Slide open the battery door on the back of the device.
- 2 Install the batteries provided with the device. Follow the diagram inside the battery compartment. For correct polarity, the springs should align with the negative sign on the battery.
- 3 Slide the battery door closed.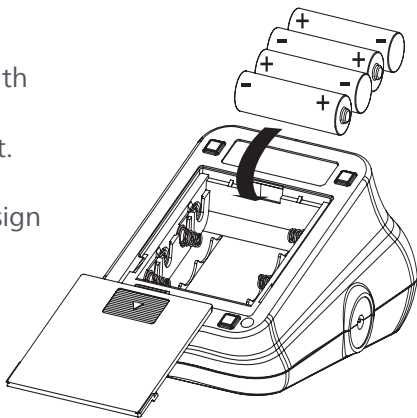


Securing the cuff

- 1 Plug the connector on the cuff tube into the device.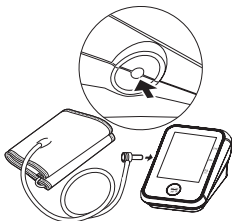

- 6 Position the cuff's edge about an inch (2-3 cm) above the elbow.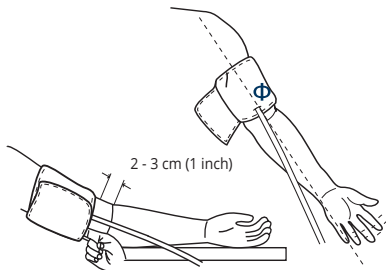

- 2 Expose your left upper arm by removing or adjusting clothing and jewelry. Make sure blood flow is not constricted by a rolled up sleeve.

- 7 Align the Φ marker (located to the right of the tube exit) with the center of your arm.

- 3 Open the cuff and loosen fully.

- 8 Tighten the cuff evenly around your arm by pulling on the end. Make sure the Φ marker stays aligned with the center of your arm.


- 4 Orient the cuff so that the tube exits towards your hand.

- 5 Place your arm through the cuff loop, with your palm facing up.


- 9 Wrap the end of the cuff over your arm to secure it in place. Don't make it too tight- allow a finger to fit between the cuff and your arm.

Taking a measurement


- 1 Lay your arm on a table with your palm facing up. The cuff should be at the same height as your heart. Sit up straight and rest your feet flat on the ground. Make sure the tube is not kinked or pinched.
- 2 Press the START/STOP button on the device to turn it on.
- 3 Wait for the device to power on, and perform the measurement while inflating the cuff.
- 4 When the measurement concludes, results are shown on the display and the cuff automatically deflates.
- 5 Remove the cuff after it deflates.
- 6 The device automatically transmits measurement results. During this process an indicator is shown in the middle of the display.
- 7 When the measurement is successfully transmitted, the device shows the following indication before powering off.
- 8 Log into your portal to see your results today.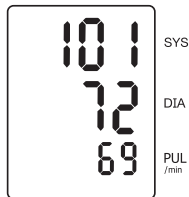

Measurements

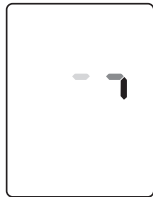Transmission
in progress

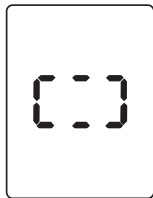Transmission
successful

Distributed by:

Telcare, LLC

600 West Ridge Road Suite 200

Linwood, PA 19061

Phone:

(888) 666-4530 (US /EE.UU.)

(1) (978) 610-4688 (International)

Monday - Friday, 9AM - 9PM EST

Email:

VCM-customer@philips.com

www.philips.com/virtualcaremanagement

The Terms of Service (“Terms”) outline the terms regarding your use of our products and can be found at www.philips.com/virtualcaremanagement/terms. If you do not agree with these Terms, do not register or use any of the Services.

© 2023 Koninklijke Philips N.V. All rights reserved.

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Telcare, LLC

600 West Ridge Road Suite 200

Linwood, PA 19061

Teléfono:

(888) 666-4530 (US /EE.UU.)

(1) (978) 610-4688 (Internacional)

Lunes - viernes, 9AM - 9PM hora del este

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VCM-customercare@philips.com

www.philips.com/virtuallcarenagement

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Los términos de servicio ("Términos") describen los términos de uso de nuestros productos y se pueden encontrar en www.philips.com/virtuallcarenagement/terms. Si no acepta estos Términos, no se registre ni use ninguno de los servicios. © 2023 Koninklijke Philips N.V. All rights reserved.