The Philips logo is located in the top left corner. It consists of the word "PHILIPS" in a bold, blue, sans-serif font. The text is contained within a white rectangular area that has a rounded bottom-right corner. A thick blue horizontal bar is positioned below the white area, extending from the left edge of the white box to the right edge of the rounded corner.

**PHILIPS**

**Instructions included**

# Take your first test today

## Installing the batteries


*If this is your first time using the device:*

- 1 Open the battery cover on the bottom of your scale.
- 2 During Install the batteries provided with the device. Follow the diagram inside the battery compartment. For correct polarity, the springs should align with the negative side on the batteries.
- 3 Replace the battery cover until closed and secure.


## Taking a measurement

- 1 Place the scale on a flat and hard surface.
- 2 Step on the scale and remain still until the measurement is complete.
- 3 The device automatically transmits the measurement results. During this process, an indicator is shown in the middle of the display.
- 4 When the measurement is successfully transmitted, the device shows the following indication before turning off.
- 5 Log into your portal to see your results today.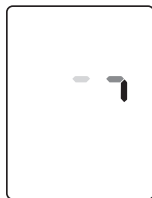

Transmission  
in progress

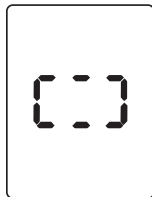Transmission  
successful

## **Distributed by:**

Telcare, LLC

600 West Ridge Road Suite 200

Linwood, PA 19061

## **Phone:**

(888) 666-4530 (US /EE.UU.)

(1) (978) 610-4688 (International)

Monday - Friday, 9AM - 9PM EST

## **Email:**

VCM-customer@philips.com

[www.philips.com/virtualcaremanagement](http://www.philips.com/virtualcaremanagement)

The Terms of Service (“Terms”) outline the terms regarding your use of our products and can be found at [www.philips.com/virtualcaremanagement/terms](http://www.philips.com/virtualcaremanagement/terms). If you do not agree with these Terms, do not register or use any of the Services.

© 2024 Koninklijke Philips N.V. All rights reserved.

The Philips logo is located in the top-left corner of the page. It consists of the word "PHILIPS" in a bold, blue, sans-serif font, positioned above a thick blue horizontal bar. The bar has a rounded bottom-right corner.

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