Connected Blood Glucose Monitoring System

Quick Start Guide


Before you start

- Before first use, charge the Philips Connected Blood Glucose Meter (BGM) for at least 6 hours with the AC charger included in your kit.
- Conduct a Control Solution Test per instructions in Section D.2 of the BGM User Manual.
- Wash your hands and testing site with warm water and soap. Dry your hands well.
- Clean and disinfect your lancing device after every use. Clean and disinfect your meter at least once a week. Please refer to details in the manual.

Take a blood glucose test




- Remove the lancing device cap by snapping it off at an angle. Insert a new lancet into the lancing device and twist off its plastic cap.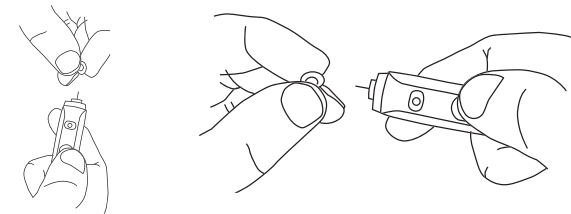


Important: The lancing device is intended for single patient use.

Blood glucose test *continued*

- Place the lancing device cap back on the device. Adjust the cap to your desired puncture depth level.
- Using your thumb, push the sliding button back. You will hear a click, indicating that the lancet carrier is locked into position.
- Power on and insert a new test strip until it is fully seated and **Apply Blood** appears.

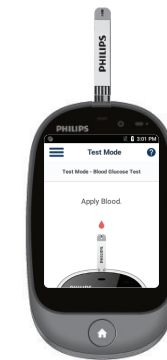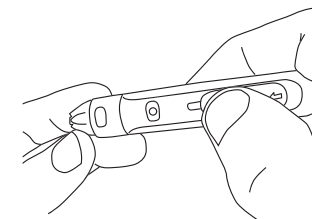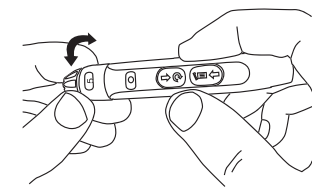
- Position the end of the device against the side of the fingertip. Press the trigger button to actuate the lancet.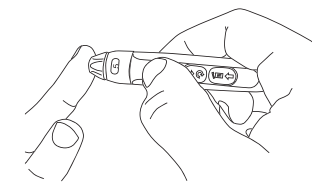

Blood glucose test *continued*

- Make sure the **Apply Blood** screen is displayed. Touch the blood drop until the end is filled. After the test strip is filled, the device will display a countdown while calculating the reading.
- Your test result will appear after the countdown is over. Select a Reading Tag during or after the countdown. Press the **CONTINUE** button to confirm your selection and send your result.

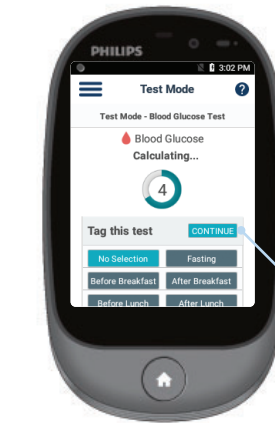
- The BGM will automatically send the results to the server. A response message from the server will be displayed if the results are sent successfully. Remove the test strip.

View & add logs

- On the Application Menu, press the **LOGS** icon OR use the buttons to access the logs.
- Press the **FILTER** button to only view certain types of logs.
- Swipe Up/Down OR use the buttons to scroll through stored readings and other entries.
- Press the **ADD LOG** button on the **+** Home Screen OR Logs Screen to create a new log entry.

View & add logs *continued*

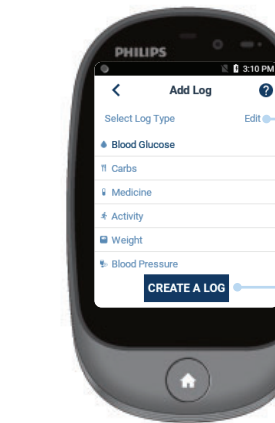- Press the **Edit** button to create or delete new types of logs.
- Press the **CREATE A LOG** button to proceed to the screen where you can enter the value, tag and timestamp a log entry.
- Press the **SUBMIT** button to save and upload the new log entry to the server.

Telcare, LLC
 600 West Ridge Road, Suite 200
 Linwood, PA 19061, USA
 Phone: (888) 666-4530 (US) or
 (1) (978) 610-4688 (International)
VCM-customer@philips.com
www.philips.com/virtualcaremanagement

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